
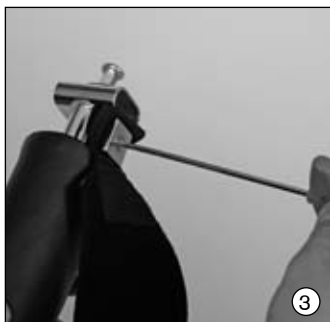
RR0201100004

DE	Haltegurt für Freistehbarren	3
EN	Patient Belt for Standing Frame	6
IT	Cintura di fissaggio per apparecchio di statica	9
ES	Cinturón de sujeción para bipedestador con barra.....	12
SV	Fixeringsbälte för ståstativ	15

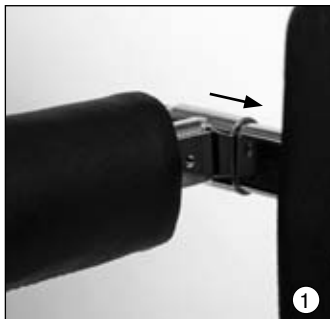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CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Information regarding operation. Information for service personnel.

INFORMATION

Date of the last update: 2013-09-02

- Please read this document carefully.
- Follow the safety instructions.

INFORMATION

Prior to mounting the patient belt, the upper body support in conjunction with the support bar and pad must be assembled correctly. Please proceed as described in the Instructions for Use 647H412 of the Standing Frame and of the support/pad accessories.

When positioning the patient and inserting the support bar and pad as well as the upper body support, proceed as described in the Standing Frame's Instructions for Use 647H412.

1 Description

1.1 Intended Use

The patient belt supports the holding of persons who are unable to walk or have a walking impediment while they are standing in the Standing Frame (see Instructions for Use 647H412).

1.2 Applications / Indication

The patient belt is used in combination with the upper body support accessory for the Standing Frame. It is appropriate for all patients who are unable to independently stand upright in the Standing Frame.

1.3 Functionality

The patient belt allows to optimally fix patients with upper body instability to the upper body support. Fixing eyelets integrated into the patient belt serve to attach the belt to the support retainer with screws.

2 Handling

2.1 Assembly

Separate the belt parts prior to attaching the belt.

Loosen the first round support from the support retainer and laterally slide down (Fig. 1).

Double-lay the fixing eyelet of the patient belt (Fig. 2), hold it over the screw hole and fix the first belt part between the support and holding plate with an Allen head screw (Fig. 3).

Mount the other belt part correspondingly. Finally, reattach the supports to the support retainer.

2.2 Adaptation

CAUTION

Risk of injury as a result of incorrectly mounted belt. Check the correct fit of the patient belt und secure hold of the hook and loop closure before the patient stands up.

- Position the patient fully upright in the Standing Frame and insert the support bar with pad/upper body support in the patient's back.
- Open the belt and adjust the length so that a space of two fingers remains between the belt and patient. The correct length can be adjusted with the hook and loop closure (Fig. 4).

2.3 Instructions for Use

Application instructions:

CAUTION

Risk of injury as a result of improper application. Do not apply the belt on open wounds or naked skin.

NOTICE

Risk of damage as a result of improper use. Do not use the belt for showering and do not sterilize. In case of wear and tear do not repair but replace with a new belt.

Washing instructions:

- Please close the belt prior to washing. In wet or moist condition, the belt's closing strength is reduced.
- Full function will be regained after drying.

3 Liability

The manufacturer's warranty applies only if the product has been used under the conditions and for the purposes described. The manufacturer recommends that the product be used and maintained according to the instructions for use.

The manufacturer is not responsible for damages caused by components and spare parts not approved by the manufacturer.

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Otto Bock Mobility Solutions GmbH