Evolight – 476C00=SK350

| | |
|---|----|
| DE Gebrauchsanweisung | 3 |
| EN Instructions for Use | 7 |
| FR Notice d'utilisation..... | 11 |
| IT Istruzioni d'uso | 15 |
| ES Manual de instrucciones | 19 |
| PT Manual de instruções..... | 23 |
| NL Gebruiksaanwijzing | 27 |
| SV Bruksanvisning | 31 |
| DA Brugsanvisning | 35 |
| NO Bruksanvisning | 39 |
| FI Käyttöohje | 43 |
| PL Instrukcja obsługi | 47 |
| HU Használati utasítás..... | 51 |
| CS Návod k použití | 55 |
| TR kullanım kılavuzu | 59 |
| EL Οδηγίες χρήσης..... | 63 |
| RU Инструкция по использованию | 67 |

INFORMATION

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INFORMATION

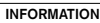
Date of the last update: 2014-03-25

- Please read this document carefully.
- Follow the safety instructions.

Explanation of Symbols

 Warnings regarding possible risks of accident or injury.

 Warnings regarding possible technical damage.

 Additional information on the fitting / use.

INFORMATION

Please read the Instructions for Use carefully. Pay special attention to the safety instructions!

1 Description

1.1 Intended Use

The wheelchair seat cushion Evolight is used for decubitus prophylaxis and therapy in patients with paraplegia, hemiplegia, geriatrics, neurological deficits, and rheumatic as well as neural and muscular diseases. It is intended **exclusively** for use in a wheelchair.

For hygienic reasons, re-use of the seat cushion or use by another patient is not recommended by the manufacturer.

1.2 Field of application

The Evolight seat cushion can be used without restrictions for patients up to a body weight of 125 kg. Use in patients up to a body weight of 180 kg is categorically possible, however prior consultation with the treating therapist is recommended.

CAUTION

Risk of redness of the skin/decubitus when exceeding the maximum permissible patient weight. When exceeding the maximum permissible patient weight, we highly recommend that the user test the cushion for a few hours under medical supervision, during which time it should be determined that no reddening of the skin occurs, before using the cushion on a permanent basis.

1.3 Contraindications

Relative contraindication: If redness of the skin occurs, stop using the seat cushion immediately and consult the treating physician or therapist.

1.4 Safety Instructions

INFORMATION

Seat cushions may only be selected and adapted by appropriately trained and qualified personnel.

⚠ CAUTION

Risk of tipping. Inserting a cushion into a wheelchair may change the wheelchair's centre of gravity, which can cause the wheelchair to tip over backwards in some cases and may result in injuries. It is therefore important to check whether anti-tippers or adjustable axle supports, such as those used for lower limb amputees, should be mounted to the wheelchair to improve stability.

⚠ CAUTION

Risk of pressure sores. All Ottobock cushions have been designed to reduce pressure sores. A cushion alone, however, cannot eliminate sitting pressure or possible pressure sores completely. Ottobock cushions are not intended to replace thorough skin care, good nutrition, hygiene and regular pressure relief.

⚠ CAUTION

Risk of redness of the skin. Before using a cushion on a permanent basis, we highly recommend that the user test it for a few hours under medical supervision, during which time it should be determined that no reddening of the skin occurs. Examine the skin for redness on a regular basis. Redness of the skin is a clinical indicator for tissue damage. Should redness of the skin occur, stop using the cushion immediately and consult a physician or therapist.

NOTICE

Risk of damaging the surface of the cushion. Do not allow sharp objects to come into contact with the cushion and do not expose the cushion to excessive heat or open flames.

1.5 Function

Thanks to its cube-like structure, the Evolight seat cushion for wheelchairs contributes to enhanced pressure distribution. The shearing forces which arise in normal foam cushions in particular are effectively reduced. Horizontally placed ventilation channels improve the removal of moisture.

Two layers of spacer fabric (one layer above the cubic structure, the other layer sewn into the cover) ensure very good air circulation and the effective removal of moisture, so that the sitting climate is significantly improved. The combination of both materials makes the Evolight one of the lightest anti-decubitus seat cushions in the world.

1.6 Accessories

The seat cushion can be combined with an incontinence cover (order number 477C00=ST410/...411/...412/...413).

2 Handling

2.1 Positioning

INFORMATION

Pressure on the cushions can only be optimally distributed, if the footrest of the wheelchair has been adjusted in such a way that the thigh of the user is in full contact with the cushion surface.

2.2

Always make sure that the cushion is correctly positioned in the wheelchair and that the cover is correctly re-mounted after it has been removed.

The spacer fabric lies above the cubic structure; the rounded side faces the rear. The cushion should be fixed to the wheelchair's seat cover with a hook and loop strip.

2.3 Cleaning and Care

NOTICE

Risk of damage due to improper cleaning. Do not use any bleaching agents. Do not tumble-dry the foam parts.

INFORMATION

Should it be necessary to disinfect the cushion (cover/foam parts), add a standard hygiene detergent during washing (e.g. Sagrotan laundry hygiene detergent).

In case of heavy incontinence, an additional incontinence cover should be used. Please observe the care and cleaning instructions for each incontinence cover used.

Cover

1. For separate washing, open the zipper on the cover and remove it.
 2. Close the zipper on the cover before washing.
 3. The cover can be washed at 65 °C using a mild, environmentally friendly detergent. **Recommendation:** Wash the cover on the gentle cycle at 40 °C in order to prevent excessive wear.
 4. Allow to air dry. When using an automatic clothes drier, select the gentle cycle at low temperature.
- Replace the cover in case of advanced wear and tear.

Foam parts

All foam parts can be washed by hand at 40 °C using a mild, environmentally friendly detergent. Allow to air dry.

3 Technical Data

3.1 Available Sizes

| | | Seat width | | | | | | | | | | | | | | | | | | |
|------------|----|------------|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|
| | | 26 | 28 | 30 | 32 | 34 | 36 | 38 | 40 | 42 | 44 | 46 | 48 | 50 | 52 | 54 | 56 | 58 | 60 | 62 |
| Seat depth | 28 | | | | | | | | | | | | | | | | | | | |
| | 30 | | | | | | | | | | | | | | | | | | | |
| | 32 | | | | | | | | | | | | | | | | | | | |
| | 34 | | | | | | | | | | | | | | | | | | | |
| | 36 | | | | | | | | | | | | | | | | | | | |
| | 38 | | | | | | | | | | | | | | | | | | | |
| | 40 | | | | | | | | | | | | | | | | | | | |
| | 42 | | | | | | | | | | | | | | | | | | | |
| | 44 | | | | | | | | | | | | | | | | | | | |
| | 46 | | | | | | | | | | | | | | | | | | | |
| | 48 | | | | | | | | | | | | | | | | | | | |
| | 50 | | | | | | | | | | | | | | | | | | | |
| | 52 | | | | | | | | | | | | | | | | | | | |
| 54 | | | | | | | | | | | | | | | | | | | | |
| 56 | | | | | | | | | | | | | | | | | | | | |

3.2 Basic Data

| | Seat cushion thickness* | Weight* | Permissible patient weight | Reference size* |
|-----------------|---|--------------------------------------|----------------------------|------------------------------------|
| Evolight | 6 cm | approx. 850g to 900g, size dependent | 100 kg | 40 x 40 cm (15.7 x 15.7 inches) |
| | Structure of the seat cushions | | | |
| | Core - PUR foam with fitted spacer fabric out of 100 % PES | | | |
| | Cover - Blended fabric out of cotton / Trevira with spacer fabric (100 % PES) | | | |

* in ready-to-use condition

3.3 Spare parts

Cover for Evolight 477C00=SV351

Evolight foam core 477C00=ST352

Spacer fabric insert 477C00=ST353

Please specify your seat width and seat depth when ordering spare parts.

4 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

The manufacturer is not responsible for damages caused by components and spare parts not approved by the manufacturer. Repairs must be carried out exclusively by authorised dealers or by the manufacturer.

5 CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock Korea HealthCare Inc.
South East Asia Co., Ltd.
Otto Bock HealthCare GmbH
Otto Bock Mobility Solutions GmbH