Cubic Foam 476C00=SK100/Combi Foam 476C00=SK200/ Contur Gel 476C00=SK300

| | |
|--|----|
| DE Gebrauchsanweisung | 3 |
| EN Instructions for Use | 8 |
| FR Instructions d'utilisation | 13 |
| IT Istruzioni per l'uso | 18 |
| ES Instrucciones de uso | 23 |
| PT Manual de utilização | 28 |
| NL Gebruiksaanwijzing | 33 |
| SV Bruksanvisning | 38 |
| FI Käyttöohje | 43 |
| PL Instrukcja użytkowania | 48 |
| CS Návod k použití | 53 |
| TR Kullanma talimatı | 58 |

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CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Information regarding operation. Information for service personnel.

INFORMATION

Date of the last update: 2014-05-14

- Please read this document carefully.
- Follow the safety instructions.

INFORMATION

- You can request this document as a PDF file from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses).
- It is possible to increase the display size of the PDF document.
- For further questions about the instructions for use, please contact the authorised personnel who issued the product to you.

1 Description

1.1 Intended Use

The Cubic Foam, Combi Foam and Contour Gel wheelchair seat cushions are to be used **exclusively** and in dependence of the degree of severity for decubitus prophylaxis and decubitus therapy for patients with paraplegia, hemiplegia, neurological deficits, rheumatic nervous and muscle disorders, as well as for geriatric patients, and are to be used exclusively in a wheelchair. For hygienic reasons, re-use of the seat cushion or use by another patient is not recommended by the manufacturer.

1.2 Field of Application

The Cubic Foam, Combi Foam and Contour Gel seat cushions can be used by patients with a body weight of up to 150 kg /330 lbs without restrictions.

CAUTION

Risk of redness of the skin/decubitus when exceeding the maximum permissible patient weight. When exceeding the maximum permissible patient weight, we highly recommend that the user test the cushion for a few hours under medical supervision, during which time it should be determined that no reddening of the skin occurs, before using the cushion on a permanent basis.

1.3 Contraindications

Relative contraindications: Should redness of the skin occur, immediately stop using the seat cushion and consult the treating physician or therapist.

1.4 Safety Instructions

The Combi Foam, Cubic Foam and Contour Gel wheelchair seat cushions have been manufactured in accordance with state-of-the-art technology and are safe in operation. The safety of

the seat cushions has been confirmed by CE certification and the declaration of conformity. The development, construction and production of the products completely meet the requirements of Medical Device Directive 93/42/EEC.

1.4.1 General Safety Instructions

INFORMATION

Seat cushions may only be selected and adapted by appropriately trained and qualified personnel.

⚠ CAUTION

Risk of tipping. Inserting a cushion into a wheelchair may change the wheelchair's centre of gravity, which can cause the wheelchair in some cases to tip over backwards and may result in injuries. It is therefore important to check whether anti-tippers or adjustable axle supports, such as those used for lower limb amputees, should be mounted to the wheelchair to improve stability.

⚠ CAUTION

Risk of pressure sores. All Ottobock cushions have been designed to reduce pressure sores. A cushion alone, however, cannot completely eliminate sitting pressure or possible pressure sores. Ottobock cushions are not intended to replace thorough skin care, good nutrition, hygiene and regular pressure relief.

⚠ CAUTION

Risk of redness of the skin. Before using a cushion on a permanent basis, we highly recommend that the user test it for a few hours under medical supervision, during which time it should be determined that no reddening of the skin occurs. Examine the skin for redness on a regular basis. Redness of the skin is a clinical indicator for tissue damage. Should redness of the skin occur, immediately stop using the cushion and consult a physician or therapist.

NOTICE

Risk of damaging the surface of the cushion. Do not allow sharp objects to come into contact with the cushion and do not expose the cushion to excessive heat or open flames.

1.5 Function

1.5.1 Cubic Foam

Thanks to its cubic structure, the Cubic Foam wheelchair seat cushion offers enhanced pressure distribution, and is particularly effective for the type of shearing forces that occur with normal foam cushions. The cube structure also promotes the circulation of air and thus contributes to a better sitting climate.

1.5.2 Combi Foam

The Combi Foam wheelchair seat cushion consists of a combination of three different layers:

- Support layer of foam (grey): Efficiently bears user's weight.
- Relax foam (pink): Low resistance foam applies less pressure to prominent body parts (e.g. ischial tuberosity).

- Final layer of white foam: This relatively soft foam increases sitting comfort and contributes to a larger contact surface between user and cushion.

1.5.3 Contour Gel

The Contour Gel wheelchair seat cushion consists of two layers of formed foam with a final layer of Technogel®.

Thanks to the cut-away leg supports, the thighs also receive effective pressure relief.

The seat ramp built into the cushion improves positioning and reduces the risk of slipping forwards in the wheelchair. The middle layer of foam ensures that there is a large contact surface and thus provides extra pressure reduction. The final Technogel® layer reduces shearing forces.

1.6 Accessories

The seat cushion can be combined with an incontinence cover (article number 477C00=ST410/..411/..412/..413).

2 Handling

2.1 Positioning

INFORMATION

Pressure on the cushions can only be optimally distributed, if the footrest of the wheelchair has been adjusted in such a way that the thigh of the user is in full contact with the cushion surface.

Always make sure that the cushion is correctly positioned in the wheelchair and that the cover is correctly re-mounted after it has been removed.

Place the cushion into the wheelchair with the zipper oriented towards the side or the rear.

The loop strip sewn to the cushion serves to fix it in the wheelchair. Place the cushion into the wheelchair with the loop strip facing the upholstery.

2.2 Assembly

For cleaning it may be necessary to remove the foam inserts from the cover. To do so, open the zipper of the cover and remove the foam parts.

For reassembly, please proceed in reverse order paying attention to the correct order:

- Cubic Foam: The cube structure faces up.
- Combi Foam: The white, soft foam layer faces up.
- Contour Gel: The Technogel® cover faces up, the small lateral face is at the rear, and the high lateral face at the front (seat ramp).

2.3 Cleaning and Care

NOTICE

Risk of damage due to improper cleaning. Do not use any bleaching agents. Do not tumble-dry the foam cushions.

INFORMATION

Should it be necessary to disinfect the cushion (cover/foam parts), add a standard hygiene detergent during washing (e.g. Sagrotan laundry hygiene detergent).

In case of heavy incontinence, an additional incontinence cover should be used. Please observe the care and cleaning instructions for each incontinence cover used.

Cover

1. For separate washing, open the zipper on the cover and remove it.
 2. Close the zipper on the cover before washing.
 3. The cover can be washed at 65°C using a mild, environmentally friendly detergent. **Recommendation:** Wash the cover on the gentle cycle at 40°C in order to prevent excessive wear.
 4. Allow to air dry. When using an automatic clothes drier, select the gentle cycle at low temperature.
- Replace the cover in case of advanced wear and tear.

Foam Cushion

Hand-wash the foam cushion at 40°C / 104°F using a mild, environment-friendly detergent. Allow to air dry.

3 Technical Data

3.1 Available Sizes

Contour Gel is only available in the sizes designated with a gray-coloured box.

Upon request, other sizes of Cubic Foam and Combi Foam are available.

| | | Seat width in cm | | | | | | | | | | | | | | | | | |
|------------------|----|------------------|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|
| | | 26 | 28 | 30 | 32 | 34 | 36 | 38 | 40 | 42 | 44 | 46 | 48 | 50 | 52 | 54 | 56 | 58 | 60 |
| Seat depth in cm | 28 | | ■ | ■ | | | | | | | | | | | | | | | |
| | 30 | | ■ | ■ | | | | | | | | | | | | | | | |
| | 32 | | ■ | ■ | ■ | ■ | | | | | | | | | | | | | |
| | 34 | | | | ■ | ■ | | | | | | | | | | | | | |
| | 36 | | | | ■ | ■ | ■ | ■ | | | | | | | | | | | |
| | 38 | | | | | | ■ | ■ | ■ | | | | | | | | | | |
| | 40 | | | | | | | ■ | ■ | ■ | ■ | | | | | | | | |
| | 42 | | | | | | | | ■ | ■ | ■ | ■ | | | | | | | |
| | 44 | | | | | | | | | | ■ | ■ | ■ | ■ | | | | | |
| | 46 | | | | | | | | | | | ■ | ■ | | | | | | |
| | 48 | | | | | | | | | | | | ■ | ■ | | | | | |
| | 50 | | | | | | | | | | | | | | ■ | ■ | ■ | ■ | |
| | 52 | | | | | | | | | | | | | | | ■ | ■ | ■ | ■ |
| | 54 | | | | | | | | | | | | | | | | ■ | ■ | ■ |

3.2 Basic Data

| | Seat cushion thickness* | Weight* | Permissible patient weight | Reference size* |
|--------------------|---|---------|----------------------------|-----------------|
| Cubic Foam | 5 cm | 650 g | 150 kg /330 lbs | 40 x 40 cm |
| Combi Foam | 5 cm | 600 g | | |
| Contour Gel | Front: 10 cm Rear: 6 cm | 1100 g | | |
| | Structure of the seat cushions | | | |
| Cubic Foam | Foam: B75N PUR ester; colour: dark grey Cover: decor fabric, black | | | |
| Combi Foam | Support layer of foam: B75N; colour: grey Relax foam: B60RL; colour: pink Final foam layer: B1625; colour: white Cover: decor fabric, black | | | |
| Contour Gel | Foam wedge: B75N; thickness: 70 mm tapering to 30 mm; colour: dark grey Support layer of foam: B75N; thickness: 30 mm; colour: dark grey Final layer: TechnoGel 130; thickness: 1.5 mm; colour: white Cover: decor fabric, black | | | |

* ready-to-use state

4 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

The manufacturer is not responsible for damages caused by components and spare parts not approved by the manufacturer. Repairs must be carried out exclusively by authorised dealers or by the manufacturer.

5 CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock Korea HealthCare Inc.
South East Asia Co., Ltd.
Otto Bock HealthCare GmbH
Otto Bock Mobility Solutions GmbH