













CE

**Nurmi**

*Instructions for Use* ..... 9

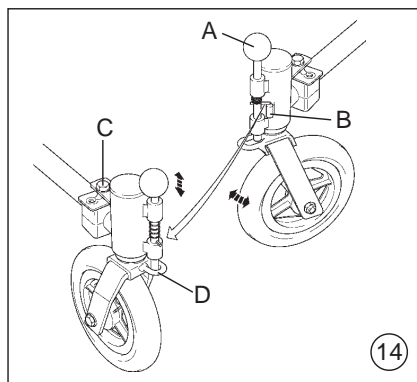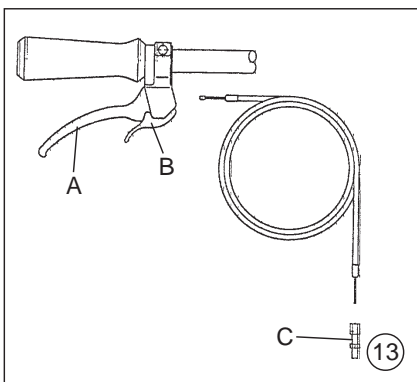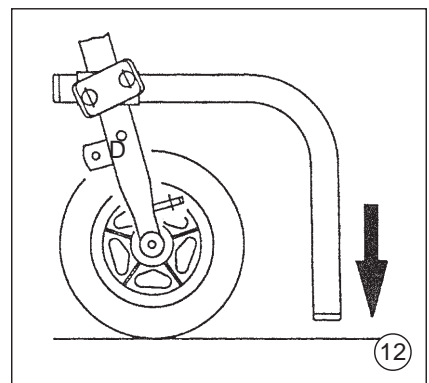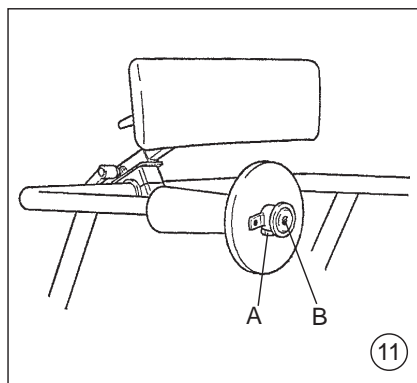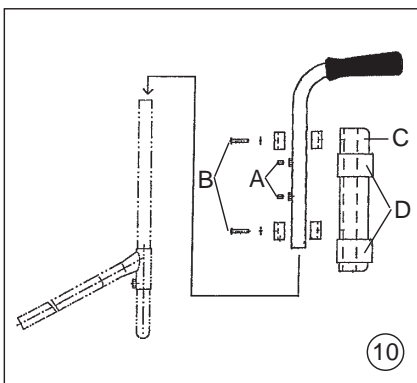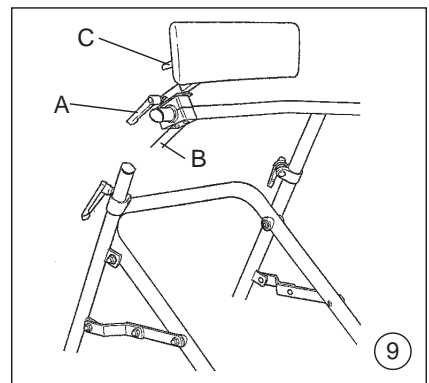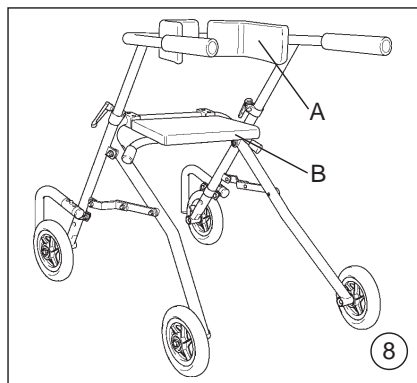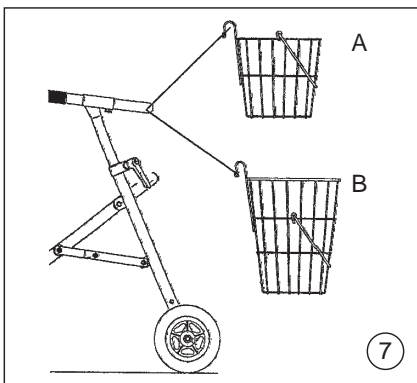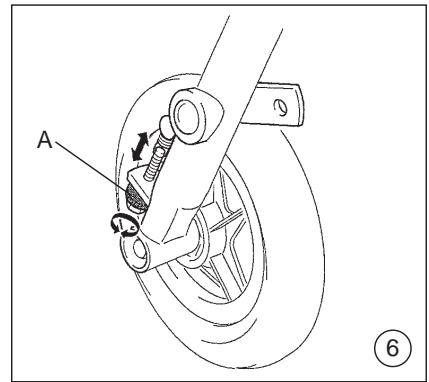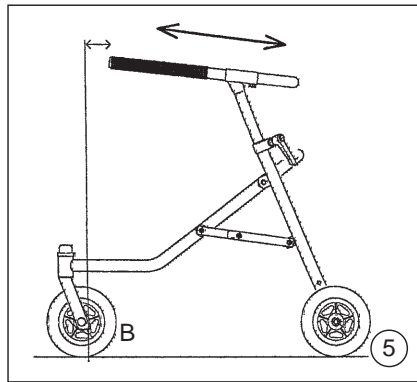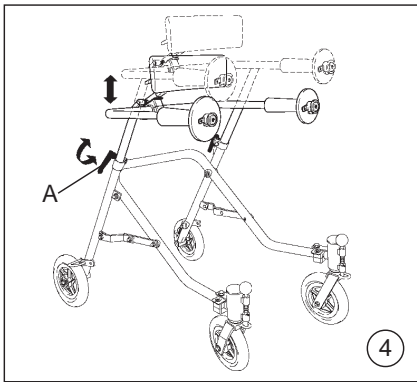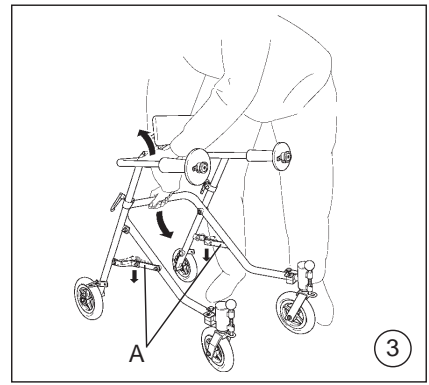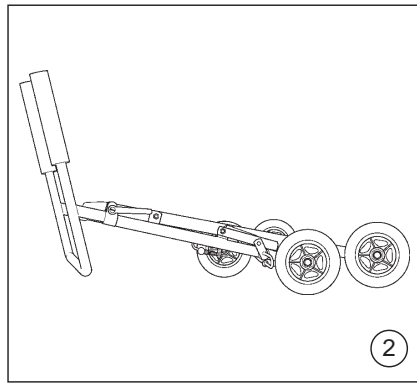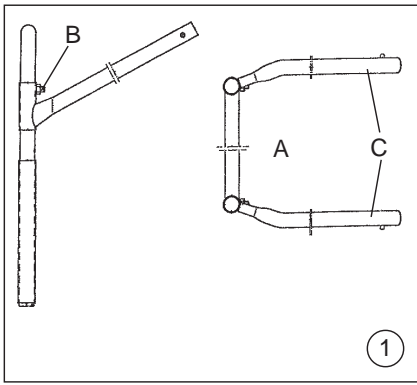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	<b>Page</b>
<b>Contents</b>	
<b>Preface</b> .....	<b>10</b>
<b>Indications</b> .....	<b>10</b>
<b>Declaration of Conformity</b> .....	<b>10</b>
<b>Set-up and use of the basic version</b> .....	<b>10</b>
<b>Unpacking and setting up</b> .....	<b>10</b>
<b>Possible adjustments and adaptations</b> .....	<b>11</b>
<b>Accessories:</b>	
Carrying basket .....	11
Hip pads .....	11
Posterior pad .....	11
Wheel-lock system .....	11
Extra-high grab rails with padded forearm rests and hand grips) .....	11
Hand plates .....	12
Anti-tip bracket .....	12
Swivel lock .....	12
Friction brake .....	12
Seat .....	12
<b>Practical notes on safety and care</b> .....	<b>12</b>
<b>Guarantee</b> .....	<b>13</b>
<b>Technical data</b> .....	<b>13</b>

## Preface

Thank you for selecting the NURMI Ergonomic Walker. Our engineers have designed the high-quality NURMI to make your life safer and easier, and we've included this manual to help you use and care for this durable Otto Bock product.

Please read the following instructions to make sure you use this product safely and for its recommended use. If you have any further questions, or if you have any problems, please contact your specialist dealer. We hope that the NURMI meets your expectations and helps to improve your quality of life.

## Indications

(Source: Catalogue of aids and list of aids for the health insurance funds, FRG)

The NURMI Ergonomic Walker helps children and adolescents with a walking handicap to offset the reduced load capacity or functional capacity of the lower extremities by making use of the strength and support available in their upper extremities.

The main aim is learning the sequence of movements involved in walking and thus widening the previously restricted range of action.

It is indicated for walking training in the case of diseases or injuries that involve a disturbance in the locomotor system and imbalance / asynergia at the same time.

The design promotes an active upright stance when standing and walking, and in contrast to traditional walkers presents itself as a conception for natural motion. For the user, it can considerably simplify the process of learning to stand and to walk and to develop his/her sense of balance.

**No restrictions as to the group of users have been encountered so far. However, should any problems occur, the guidance of a medical specialist or orthopedic surgeon should be obtained in any case.**

## Declaration of conformity

Otto Bock HealthCare GmbH declares as the manufacturer with sole responsibility that this product is in conformity with the requirements of Directive 93 / 42 / EEC.

## Set-up and use of the basic version

NURMI is a "posterior walker", which means that unlike an anterior walker, which the user pushes from behind, it is designed to be pulled from a position in front of it (the user 'drags' the walker). Thus an optimum freedom of forward movement for the legs is ensured.

Since children instinctively tend to lean and fall backwards, NURMI is equipped with reverse-roll locking devices integrated into the rear wheels, thus providing optimal support.

The references made in these instructions to graphics and items in the illustration section are to be read as follows: **(1/A)** = Fig. No. 1 / item A

## Unpacking and setting up

In the original package you will find the following components:

- Basic frame, collapsed
- Grab rail, collapsed
- Any accessories ordered (partly pre-assembled)

In order to set up the NURMI Ergonomic Walker, please proceed as follows:

- Please carefully remove the transport safety snaps and any packaging material.
- Now unfold the grab rail. Place the insert tubes **(1/A)** in a parallel position to each other and slightly tighten the stud screws **(1/B)** just as much as necessary for preventing the insert tubes from twisting.
- Then slot the grab rail into the "Walker" **(2)** which remains collapsed. While sliding in the grab rail, press the spring-loaded buttons **(1/C)** inside (extension stop).  
Insert the grab rail into the frame until the extension stops on the left and right locks.
- Place NURMI on its rear wheels and unfold the front part of the frame as shown in illustration **(3)**.
- Make sure that the folding locks **(3/A)** are locked; if necessary, press them again To make sure they engage.
- Now set the grab rail to the height which is most convenient for the patient, and then tighten the two clamping levers **(4/A)**.

**For ease of operation, draw the clamping lever a little bit away from the screw; it may then be moved off-load.**

**Note: There is an extension stop that limits the maximum height setting; you must not attempt to extend it past this point.**

- Finally, please loosen the stud screws **(1/B)** and adjust the grab rail in such a way that the front edge of the grab rail ends approximately **7.5 cms** behind an imaginary vertical line starting at the point where the front wheels touch the ground on an even surface. **See illustration (5).**

**Note: This setting of the grab rail depth must be checked and, if necessary, reset after each height adjustment, as otherwise there may be a danger of tipping forward.**

## Possible adjustments and adaptations

### Height and depth of grab rail

As described in "Unpacking and setting up".

### Folding for transport

Before folding, please loosen the folding locks **(3/A)** by pushing them from a stretched into a slightly buckled position. Please proceed by carrying out the steps described in "Unpacking and setting up" in reverse order.

### Caution - Danger of pinching:

**While folding the walker, do not hold on to other parts than the ones described, especially not the folding mechanism.**

## Accessories

**Any accessories that have not been installed by the manufacturer must be installed by trained staff.**

### Carrying basket

There is a carrying basket available fit each of the NURMI models.

Model 2413 **=(7/A)**, Model 2420-31 **=(7/B)**

Apart from its practical function, its therapeutic use is to provide a stimulus for the patient to collect objects.

The basket has two hooks welded on and can simply be hinged onto the grab rail.

### Hip pads

The hip pad **(8/A)** provides lateral support for the patient from the thigh to the pelvis area.

By tightening the retaining screws the hip pad width can be adjusted; thus it is also safe from unwanted twisting. A hose part has been inserted for additional frictional drag. For this reason, the inner diameter of the plastic clamps is slightly larger than that of the grab rail tube.

### Wheel-lock system

The wheel-lock system enables the patient both to regulate braking and to prevent the NURMI walker from rolling away.

To activate the wheel-lock system, squeeze the brake lever **(13/A)** until you hear the locking jaw **(13/B)** snap into place. To release the wheel-lock system, squeeze the brake lever slightly and open the locking jaw with your index finger.

Should the braking power be insufficient, adjust the relevant brake cable with the adjusting screw **(13/C)**.

As with the hand plates, the original grip is replaced by a profiled sponge rubber grip.

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By means of the posterior pad **(9)** the user can be positioned within the grab rail frame in the best possible way. Since many users tend to lean backwards, this pad keeps the user at the front of the grab rail frame, in an upright position.

The posterior pad can be adjusted in three dimensions.

- depth adjustment with clamping lever at the clamping support holder **(9/A)**
- height adjustment by turning the clamping support holder **(9/B)**
- angular adjustment of the posterior shell after opening the conical clamp **(9/C)**

### Extra-high grab rail with padded forearm rests and hand grips

The extra-high grab rail with padded forearm rests and hand grips was developed for users who find it easier and safer to support themselves on their forearms rather than on their hands, with arms straightened.

The grab rail is mounted and adjusted to suit the patient as described in the introduction. After that, the grips with the padded arm rests mounted on them are inserted into the ends of the grab rail. The two stud screws **(10/A)** prevent the grips from getting out of place or turning. After loosening the retaining screws **(10/B)** and the plastic clamps the padded forearm rests **(10/C)** can be moved along the rail grip; due to the asymmetric screwing, the forearm rests can be mounted in a twisted position, too.

The distance between forearm rest and grip should be adjusted to avoid pressure areas and cramped postures.

The patient's forearms are fixed by two Velcro straps **(10/D)** each.

### **Hand plates**

The hand plates provide the optimum grip position on the grab rail.

The wooden disc prevents the hands from slipping forwards, off the rail.

Adjust the position of the hand plate to the patient's needs considering the grab rail depth; secure the hand plate by fastening the stud screw **(11/A)**.

The grips that were originally glued on have to be removed first; all glue remnants must be removed using a solvent.

The open ends of the grab rail are closed with end caps **(11/B)**, which prevent the hand plates from sliding off the grab rail.

### **Anti-tip bracket**

The anti-tip bracket **(12)** prevents the walker from tipping backwards; it is available for each model size.

The anti-tip brackets are inserted into the pre-installed anti-tip bracket holders, and spring buttons prevent them from turning.

Please check that the distance between the lower edge of the anti-tip bracket and the ground does not exceed 2 cms.

### **Swivel lock**

The swivel lock is ideally suited for the early stage of the therapy with NURMI models when an increased maneuverability is not desired, since it would require substantial coordination skills of the user.

The swivel lock offers either a rigid or a steerable model according to individual needs. To release the swivel lock, simply pull the retaining bolt **(14/A)** at the ball-ended knob and turn it halfway round until the locking pin stops on top of the supporting socket **(14/B)**.

To activate the lock, please proceed in reverse order. See to it that the retaining bolt catches of the retaining latch **(14/D)**.

Loosen the retaining screw **(14/C)** and turn the swivel lock to readjust the straight-line motion.

### **Friction brake**

The function of the friction brake is to slow down the process of rolling on in a regulated way.

By loosening or fastening the knurled nuts **(6/A)**, the pressure of the friction brakes can be adjusted, thus varying the rolling speed.

**Note: Proper functioning should be checked regularly and reset, if necessary. Please replace worn or damaged wheels immediately.**

### **Seat**

The seat **(8/B)** is not designed to be a comfortable piece of furniture but to provide a simple seat which enables the user to rest briefly.

The seat is in an upright position while the patient is walking; it is simply folded down when required.

The tightness of the plastic clamps can be adjusted with the retaining screws **(8/C)**.

## **Practical notes on safety and care**

- Please observe the maximum load capacity!
- NURMI is a therapeutic appliance and may only be operated under supervision!
- All adjusting levers are designed to be tightened or fastened with your fingers. Using tools may damage or over tighten these levers.
- Attaching any heavy items to the NURMI walker may affect stability and is not recommended.
- After all adjustment work, re-fasten screws tightly.
- Clean frame, synthetic leather and plastic parts using mild household cleansers only.

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e-mail: reha@ottobock.de · Internet: <http://www.ottobock.com>

Otto Bock has been certified by the German Society for the Certification of Quality Assurance Systems (DQS) in accordance with DIN EN ISO 9001 standard, reg. no. 779 (management system)